

TIMESTRIP™

One Hour Indicator



The Convenient Visual Monitoring Aide

With a quick glance personnel can quickly identify an endoscope that is close to exceeding the one hour time requirement and prioritize cleaning, helping to avoid the need for suboptimal extended soaks.

IMPROVES workflow efficiency by easily identifying and prioritizing endoscopes approaching maximum allotted time between bedside pretreatment and manual clean.

REDUCES the risk of inefficient reprocessing methods and instances of delayed reprocessing times.

If the endoscope is not immediately cleaned after each procedure, residual organic debris will begin to solidify and it may be difficult to effectively reprocess the endoscope.

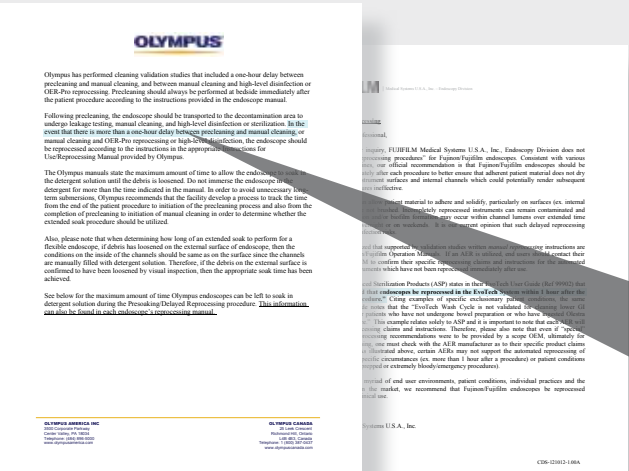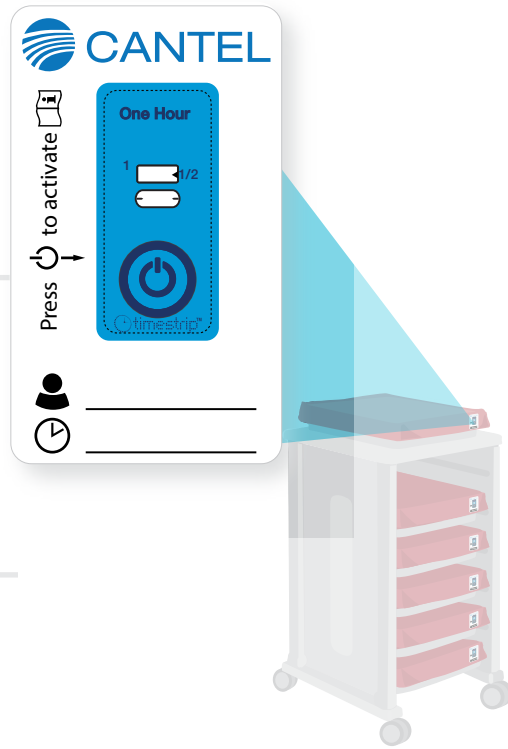


SUPPORTS best practice trends for enhanced tracking and monitoring protocols, and complies with endoscope manufacturer recommendations and societal guidelines of one hour to transition from procedure room to manual clean sink.


"In the event that there is more than a one-hour delay between pre-cleaning and manual cleaning..."
OLYMPUS ENDOSCOPY DIVISION




HOW IT WORKS

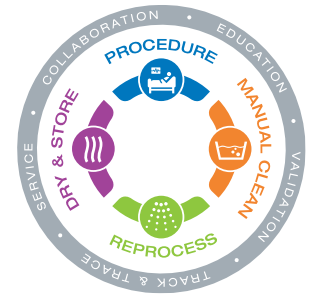


Activate TIMESTRIP™ One Hour Indicator by pressing the activation button. The time indicator signifies the product is "ON" after activation. Once activated, the indicator window fills with dye, showing the elapsed time.

- 


1 Immediately after completing the bedside pretreatment, press button to activate TIMESTRIP.

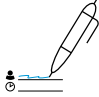- 


2 Document user information and the time bedside pretreatment was completed.

- 

3 Peel and stick TIMESTRIP One Hour Indicator on the CLEANASCOPE™ Liner to be monitored.

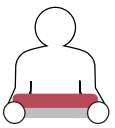- 

4 Transport the CLEANASCOPE Tray with Liner containing the endoscope, or device, to the manual cleaning sink and perform manual cleaning prior to dye completely filling white windows.

- 

5 If the dye completely fills the white window prior to cleaning, be sure to document the time lapse, then follow the manufacturer's recommendations for delayed reprocessing.

Part Number	Description
100060	TIMESTRIP™ One Hour Indicator (100 strips per box)
Consult Your Sales Representative	ENDO CARRY-ON™ Procedure Kits, CLEANASCOPE™ and CLEANASCOPE™ Advantage Storage and Transport Systems, CLEANASCOPE™ and CLEANASCOPE™ Advantage Liners

TIMESTRIP™ is a trademark of TIMESTRIP UK Limited. CLEANASCOPE™ is a trademark of Cantel (UK) Limited. ENDO CARRY-ON™ is a trademark of Medivators Inc.

MEDIVATORS IS NOW CANTEL

www.medivators.com

TO PLACE AN ORDER

p: 1.800.328.3340 (customer service) f: 1.800.686.8493 e: custserv@medivators.com

© 2019 Medivators Inc. P/N 50098-1667

