

SCOPE BUDDY™ PLUS

Endoscope Flushing Aid Quick Start Guide

BEGINNING OF THE DAY

- Install tubing sets
- Perform Daily Dosing Verification Test

DOSING VERIFICATION TEST



- Enter Manual Mode
- Press **OK**



- Enter 123 for Endoscope ID
- Select **Dosing**; Ensure 20mL is displayed
- Place open end in graduated cylinder
- Press **Play**

RUNNING SCOPES – AUTOMATIC MODE

- Scan: User ID, Scope ID, Patient ID (if applicable)



- Press **Play** to dose sink
- Press **Play** to dose clean source (if used)
- Press **Play** and begin brushing, if prompted.



- Connect scope for aspiration
- Remove biopsy cap, strainer
- Close pinch clips (if used) or block off A/W and Auxiliary Water Connector (if applicable) (DSD Hookup)



- Reattach biopsy cap, strainer
- Open pinch clips A/W connector and Auxiliary Water Connector (if applicable) from DSD EDGE™ Dual Basin AER or DSD-201 Dual Basin AER Hookup
- Begin flushing (1min 35sec) and air purge (15sec) steps

END OF DAY

- Discard all 24-hour multi-use hookup tubing

Sink Volume: _____

Detergent Solution (mL): _____

Refer to User Manual for more detailed instructions.

SCOPE BUDDY™ and DSD Edge™ are trademarks of Medivators Inc.

© 2020 Cantel Medical Corp. LIT-00270/A

