

# ENDO SMARTCAP™

Tubeing for FUJIFILM™ 700 Series GI Endoscopes

## PROPER CONNECTION


The Instructions For Use (IFU) must be followed when connecting the ENDO SMARTCAP™ Tubeing to the FUJIFILM™ 700 Series GI Endoscope.

Connect the distal tip of the ENDO SMARTCAP Tubeing to the FUJIFILM 700 Series GI Endoscope. **Carefully align the pins on the side of the endoscope port to the grooves of the ENDO SMARTCAP™ Tubeing connector.** Push the connector fully onto the port and twist the connector to the desired position. Stop when you can no longer turn the connector. Do not force it to turn.

As stated in the IFU, care should be taken to properly align the pins on the side of the endoscope port to the grooves of the ENDO SMARTCAP Tubeing Connector. When in-servicing customers, highlighting the need to properly align the pins in the grooves is important to prevent damage to the connector, which could lead to visible debris occurring on the outside of the endoscope connection.



Align pins to grooves.

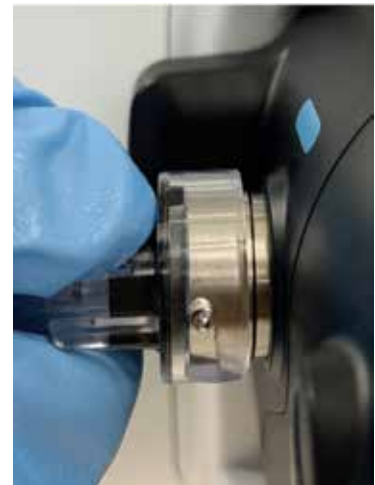

Push in and twist.

## STEPS TO AVOID DAMAGE TO THE ENDO SMARTCAP AT THE CONNECTION POINT

1. Make sure to align the pins on endoscope port into the grooves of the ENDO SMARTCAP Tubeing connector before twisting.
2. Make sure to PUSH in on the connector as you twist.
3. Use both hands to connect the ENDO SMARTCAP Tubeing; one hand should hold the connector and one hand should be used to support the endoscope while pushing in on the connector.
4. The ENDO SMARTCAP Tubeing is designed for 24-hour use only and should never be reprocessed.
5. Never attach or reattach ENDO SMARTCAP Tubeing to an endoscope that is/was inside a patient and has not been reprocessed.
6. Follow the manufacturer IFU for the proper endoscope reprocessing instructions.

ENDO SMARTCAP™ is a trademark of Medivators Inc.

All other company or product names referenced in this document are trademarks of their respective owner.

[www.medivators.com](http://www.medivators.com)

p: 1.800.444.4729 | e: [custserv@medivators.com](mailto:custserv@medivators.com)

© 2020 Medivators Inc. LIT-00202/A